


## BluTab – Waterline Maintenance Tablets

**BluTab** Waterline Maintenance Tablets are formulated specifically for dental unit waterlines. BluTab is designed to be continuously present in your water lines and helps keep lines clean.

 <p><b>BluTab 750 ml tablets 50/Bx</b> from HENRY SCHEIN Supplier Part <b>9911587</b> BluTab 750 ml tablets 50/Bx</p> <p>26.45 USD <input type="text" value="1"/> Box <a href="#">Add to Cart</a></p>	 <p><b>BluTab Waterline Tablets 2 Liter 50/bx</b> from HENRY SCHEIN Supplier Part <b>9918319</b> BluTab Waterline Tablets 2 Liter 50/bx</p> <p>37.49 USD <input type="text" value="1"/> Box <a href="#">Add to Cart</a></p>
--	--

### Directions for use:

1. Before starting a treatment program with BluTab, flush lines with Citrisil Shock Tabs. Repeat flushing every 2 months or when needed.



**Citrisil Shock Tabs 10/Bx**  
from HENRY SCHEIN  
Supplier Part **3270378**  
Citrisil Shock Tabs 10/Bx

18.79 USD  Box [Add to Cart](#)

2. Place tablet or tablets (see product insert) in empty dental unit water bottle every time you refill the water. BluTab is intended for use with potable water, distilled or filtered water (quench system) is recommended.
3. Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.
4. BluTab may be left in the waterlines overnight and for extended periods of non-use. BluTab has been shown to be effective for up to 28 days. Use water solution until it is expended; or up to 28 days. BluTab's are to be used every time you fill the bottle from the dental chair.